



Dimension : 313 x124mm

Lorem ipsum

Approved by : Phetcharat M.
ACRF No : 082/2024
Date :19/2/2024

© Love Light Glow - Natural Rubber Latex Male Condoms – TURN IT ON – RISE AND SHINE
Love Light® condoms are designed for pleasure and reliability. • Condom is a medical device. Class C - Non-Sterilized.

INSTRUCTION FOR USE
DESCRIPTION:

Natural rubber latex male condom, also known as a prophylactic, is a thin sheath used as barrier against unwanted pregnancies, STI and HIV. The intended use of the condoms is for contraception and prevention of sexually transmitted infections (STI).
Condoms do not completely eliminate the risks of pregnancy and sexually transmitted infections (STIs), if used properly, it will help to reduce the risk of transmission of HIV infection (AIDS) and many other sexually transmitted infections.

INDICATIONS: It is used as a Barrier Contraceptive for prevention of pregnancy & prevention of Sexually transmitted disease.

INSTRUCTION FOR USE: FOR MAXIMUM PROTECTION IT IS IMPORTANT TO FOLLOW THESE INSTRUCTIONS CAREFULLY.



1. Carefully tear off foil pack, making sure not to damage the condom.
2. During removal from foil, avoid damage to the condom by fingernails, jewelry, etc...



3. Squeeze the reservoir end of the condom gently, between the thumb and the forefinger to expel the air out of the tip, while placing the condom over the head of the erect penis. This must be done before any contact occurs between the penis and the prevention of sexually transmitted infectious and pregnancy.



4. While still holding the reservoir tip, unroll the condom over the entire length of the erect penis with the other hand.



5. Stop and check if the condom slipping off or tightening excessively onto the penis as this might lead to breakage.



6. After ejaculation, hold on the rim of the condom and slowly withdraw the penis while it is still erect so that the condom does not slip off.



7. Remove the condom from your penis by sliding it off slowly. It is important not to permit the penis or used condom to touch the vagine area after withdrawal.

8. Wrap the used condom and dispose it hygienically in a closed rubbish bin.


WARNINGS - IMPORTANT POINTS TO KNOW BEFORE USING A CONDOM:

Check the expiry date and do not use the condom beyond the expiry date printed on the package.

1. If extra lubrication is needed, use water-based lubricants.
2. DO NOT USE OIL BASED LUBRICANTS such as petroleum jelly, mineral oil and cold cream, as they may weaken the condom and cause subsequent failure of the product. So can some topical medicines that are applied to the penis or vagina. If you are not sure about the effect of the medicines you're using on condoms, ask your doctor or pharmacist.
3. Do not use the condom if the rubber materials are sticky or brittle. If the individual container is obviously damaged, discard that condom and use a new one from an undamaged package.
4. Seek medical assistance as soon as possible, at least within 72hour should a condom leak or burst during use.
5. Condoms are for single use only; reused condom could increase the risk of breakage and infection.
6. When used correctly every time you have sex, latex condoms help prevent pregnancy, reduce the risk of transmitting HIV/AIDS and other sexually transmitted infectious.
7. No method of contraception can provide 100% protection against pregnancy or transmission of HIV and STIs. If you suspect that you or your partner may have become pregnant or infected, consult your doctor immediately.
8. This product contains natural rubber latex which may cause allergic reactions including anaphylactic shock. If you are allergic to natural rubber latex products, consult your doctor for advice before use.
9. Store in a cool dry place away from direct sunlight.
10. Keep away from children.
11. There is a risk of choking using a condom orally.
12. Condom is a medical device to be used by lay persons - adult male or female.
13. Report any serious incident occurred on the condom to the manufacturer and the relevant national competent authority.
14. These condoms conform to International standards ISO 4074.

Model No.: Love light Glow Condom - P53S-T2-180-Glow

CAUTION: This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions Including Anaphylactic Shock if the user is allergic to latex. If you are allergic to natural rubber latex products, consult your doctor for advice before use.

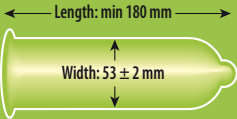

	Use by date		Store in cool, dry place
	Consult instruction for use		Keep away from sunlight
	Batch code		Do not use if Package is damaged
	Manufacturer		For single use only
	Contains Natural Rubber Latex		Medical device

Storage & Stability

Store in a cool dry place away from direct sunlight. Shelf life of Natural Rubber Latex Male Condom is 3 years.
Condom should always be stored inside the consumer package.

Importer & Distributor: M/s BONNE SAI SANTE HEALTHCARE PRIVATE LIMITED, Door No. 202/3(571/3), Room No. F1, South Facing, First Floor, Anna Salai from Teynampet Signal to Kamarajar Arangam Teynampet, Chennai-600018,
Mylapore (part 2) Mylapore Taluk, Chennai, Tamil Nadu (India) - 600018
Telephone No: 8754451519 - E-mail: sugant@wanadoo.fr
Website: www.bsshealth.in
Import License no: IMP/MD/2023/001061

Karex Industries Sdn. Bhd., PTD. 7906 & 7907, Taman Pontian Jaya, Bt. 34
Jalan - Johor, 82000 Pontian, Johor Darul Takzim, MALAYSIA.

NOT TO BE SOLD LOOSE - FOR SALE IN INDIA ONLY.
LOVE LIGHT BRANDED CONDOMS ARE AVAILABLE IN 3S AND 6S PACKS.
REV 01 / 2024
K-IFU-08(v.3)



www.bsshealth.in



Dimension : 313 x124mm



Additional information

TURN IT ON - RISE AND SHINE:

In order to glow, LOVE LIGHT condoms need to be exposed to light for about 30 seconds. You can choose to light it up on three ways: while the condom still in its wrapper; when you are putting on the condom; or after it is already on. If you exposed the condom to light it's still rolled up in the wrapper, the tip will glow better than the rest of the condom once it is unrolled. So for the best results, keep the light on bright while you put on the condom, or turn up the lights for about 30 seconds after you have put it on. When you turn off the lights, prepare for the night to remember! LOVE LIGHT will glow brightly for about 15 minutes, followed by an after glow that can last for hours. Re-exposing the condom to light while it is still on will recharge it and make it glow brightly again. But remember, you should never reuse a condom.



**LOVE
LIGHT
TECHNOSEX**

GLOW CONDOM
RISE & SHINE

LUBRICATED CONDOM

WORLD'S ONLY
GLOW CONDOM
HIGH QUALITY